


Excellent Supplier of X-ray Grids

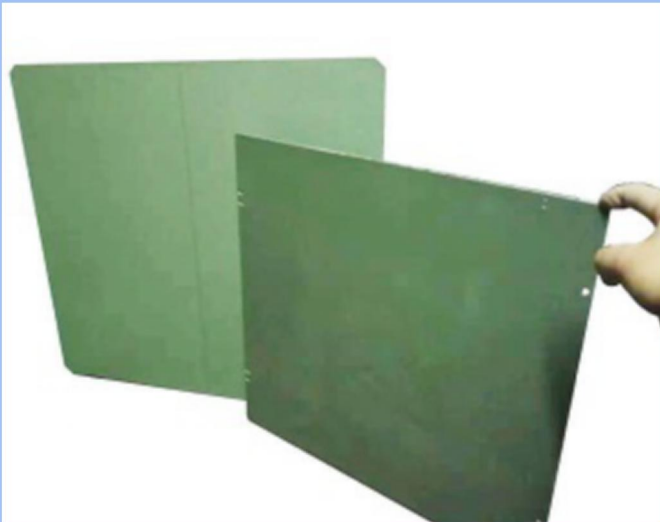

A grid is recommended when x-raying objects that are 10 cm thick or greater. When thick objects are x-rayed they create scattered radiation which contributes to fogging on the image.

Weifang Newheek Electronic Tech Co., Ltd.

Add: No. 1 Factory Building of No.13426 of Yuqing East Street, Weifang, Shandong, China.
Website: www.x-raygrid.com Tel: 86 536 8882360 Email: service@newheek.com

Quick Details

Properties : Medical X-ray Equipments & Accessories

Name : Movable X-ray Grid

Functions : X-ray protect

Origin : Newheek , China

Size : can be customized

Line : 28lp/cm

Ratio : 8:1

F.D. : 100cm or 80cm

Type: Paper-Interspaced

Absorber: Pb

Cover: Aluminum

Item : medical lead anti scatter grid X-rays

Package : Standard carton package for medical lead anti scatter grid X-rays.



Product Application

In photography, the Grid is placed between the film and the body, and the scattering rays can not pass through the gap between the lead strips because of their angle with the lead strip. Therefore,

Weifang Newheek Electronic Tech Co., Ltd.

Add: No. 1 Factory Building of No.13426 of Yuqing East Street, Weifang, Shandong, China.
Website: www.x-raygrid.com Tel: 86 536 8882360 Email: service@newheek.com

most of the scattering rays are absorbed by the lead strip, which greatly reduces the scattering rays received on the film, thus improving the contrast of the image, improving the imaging quality of the X-ray film and reducing the damage to the human body.

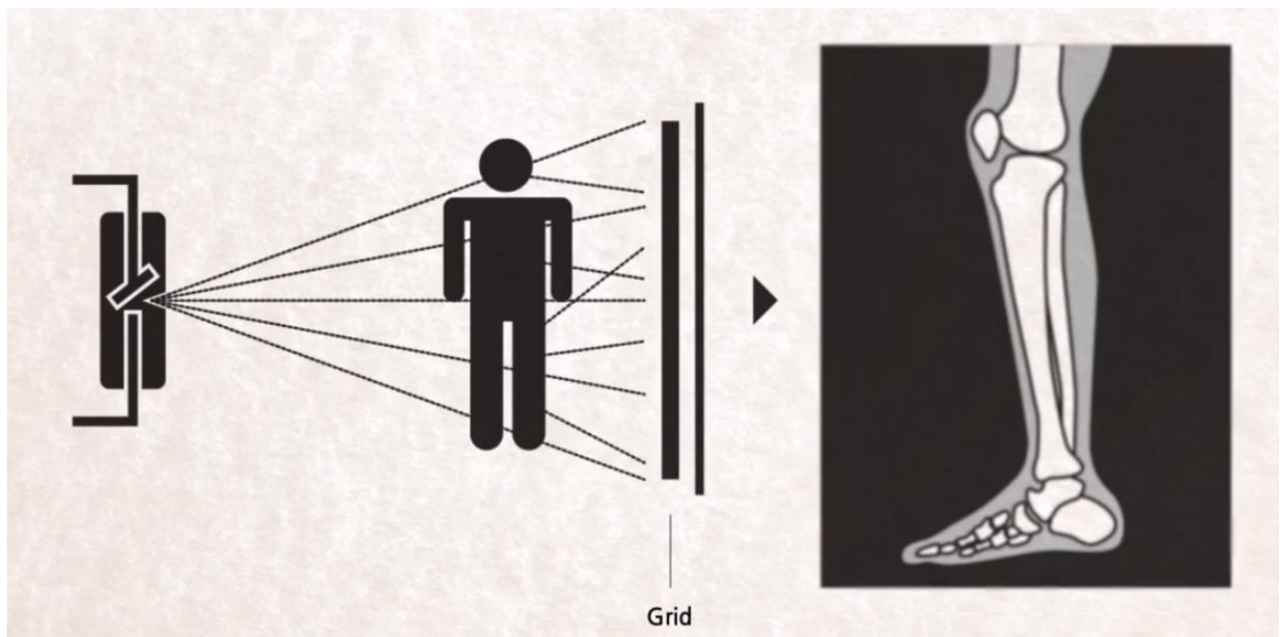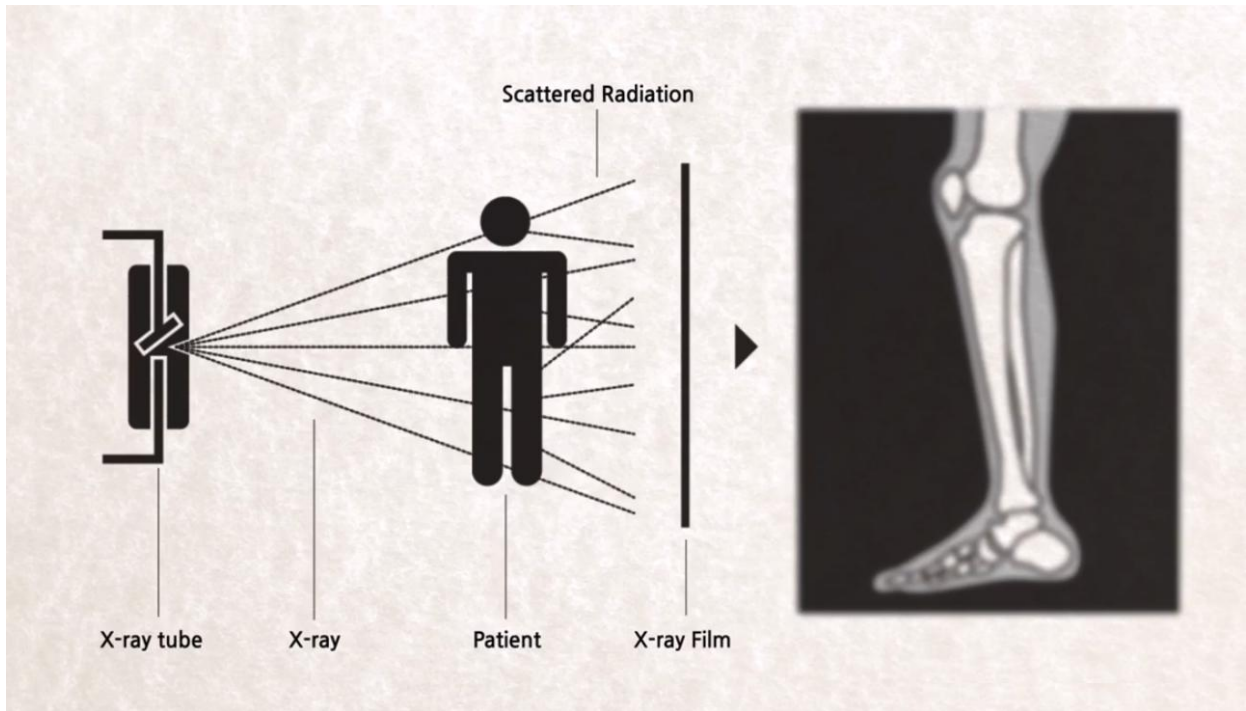

Weifang Newheek Electronic Tech Co., Ltd.

Add: No. 1 Factory Building of No.13426 of Yuqing East Street, Weifang, Shandong, China.
Website: www.x-raygrid.com Tel: 86 536 8882360 Email: service@newheek.com

Product Using

Movable X-ray Grids Newheek offered can be used for Industrial Frequency X-ray Units. Such as 200mA X-ray machine, 300 mA X-ray machine and 500mA X-ray machine. Movable X-ray Grids Newheek offered are used in vibration mode.

FAQ

The common problems on the **X-ray Grids**

1. How many types of the X-ray Grids in NEWHEEK?

Newheek operate JPI's grid and movable X-ray grid and specially customized grid.

2. What is the Payment Method ?

We could accept payment by TT, Western Union, Visa Credit Card, Paypal etc.

3. What is the interspaced material of the Grids?

Aluminum, Carbon or Fiber .

4. How long is the Lead Time?

Within 10 workdays for x ray Grids after receipt of payment.

5. Can the X-ray Grids be customized?

Of course sure, please send your request to

service@newheek.com. We will contact you as soon as possible.

Weifang Newheek Electronic Tech Co., Ltd.